

Instructions

Check your drug(s) before using if possible

Put a small amount (size of a grain of salt) of substance in a cup

Fill cup with tap or bottled water (30 ml – eg. the size of a shot glass)

Dip strip as shown for 15 seconds. Don't pass the blue line



Wait 1 full minute and read the result:

1 line:
FENTANYL
detected



2 lines:
NO Fentanyl
detected

You can check opioids, meth, cocaine, MDMA or any other substance for fentanyl

Strips **cannot** detect all the different kinds of fentanyl, including carfentanyl, and may show as negative even when present.

Always use safely even if the result is negative

If you don't see any line, redo the test

When testing stimulants it is **important to ensure enough water is used** – make sure you have used a full 30mL (the size of a shot glass) and **do not** provide more than a grain of salt sample!

More info:

<https://towardtheheart.com/update/drug-checking-services-available>
www.drugchecking.ca

Watch an online video:

https://drive.google.com/file/d/1Z6S_A_T8tjF2iri3nVanrHcIs5MgOnfK/view